LINFOPRESS EASY

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TECHNICAL FEATURES:

- MODEL: LINFOPRESS EASY.
- COMMERCIAL CLASSIFICATION: instrument for pressure therapy.
- TECHNICAL CLASSIFICATION: medical device, Class I Type BF.
- MEDICAL DEVICE CLASS: IIa (93/42/EEC directive).
- REFERENCES STANDARDS: EN 60601-1 (IEC 601-1), EN 60601-1-2 (IEC 601-1-2).
- GENERATOR'S SIZE: 208 x 306 x 130 mm.
- WEIGHT: 3 Kg.
- POWER SUPPLY VOLTAGE: ~ 230V single phase (~ 115V upon request).
- LINE FREQUENCY: 50-60 Hz.
- POWER CONSUMPTION: 180VA.
- LINE FUSES: 2 x 1,6A slow.
- NUMBER OF SECTORS: 4.
- PRESSURE REGULATION: from 0 to 160 mmHg.
- AIR COMPRESSOR CAPACITY: 20 lt/min.
- SUPPLIED PRESSURE: max. 3Atm.

• APPLICATORS AND ACCESSORIES:

- Leg strap four sectors
- Pair of legs strap four sectors
- Arm strap four sectors
- Pair of arm strap four sectors
- Four-outlet duplication set
- Two sector abdominal strap
- Splitter set for abdominal strap

- **CE** Certified: this unit fully complies to specifications given by the European Community under 93/42/EEC e 89/336/EEC Directives.

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The technical features and sizes of the appliances described herein are subject to modification, based on technological evolution and on the constant improvement of our production.

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


SEQUENTIAL OPERATION

The treatment starts when the first sector inflates (Phase 1) until the pressure previously set by the operator is reached.

Then, the second sector inflates (Phase 2). Once Phase 2 reaches its steady state, the system reaches Phase 3 and so on until the last phase is reached.

Once all the sectors have been inflated, then they start to deflate (Phase 5).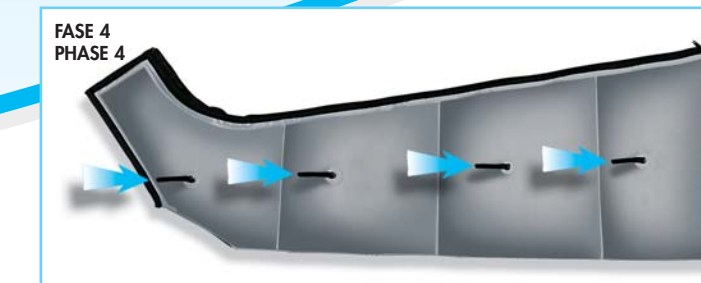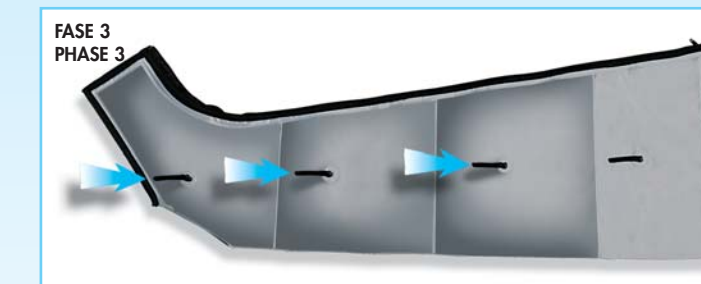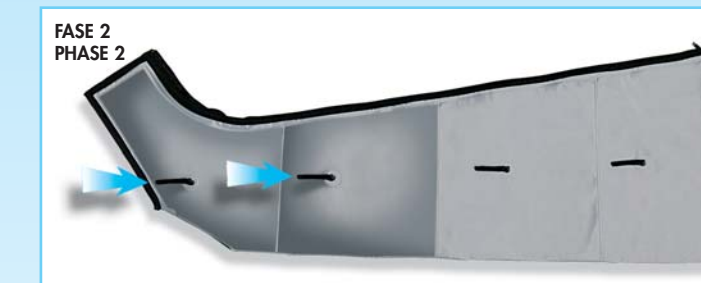

Linfopress Easy is the last novelty in sequential pressotherapy.

This type of appliances that have been used for years for medical and physiotherapy purposes can now be used by a large number of users and allows to act on the user physiologically, wherever help for problems due to an incorrect functioning of the lymphatic or venous system is needed.

Linfopress Easy has been designed and realized to comply with user's requirements for home applications as well as for professionals in physiotherapy and aesthetic applications. Application at home is particularly important in that a long and continual use offers better guarantees of long lasting results. The attractive design, compact size and ease of use makes **Linfopress Easy** suitable for home treatments.

The appliance adopts a powerful compressor which allows to obtain a 30 second inflating cycle with the possibility for the user to adjust the pressure precisely from 0 to 160 mmHg.

The inflating cycle takes place sequentially on four different sectors of the leg and the arm strap with overlapped sector inflating system. This inflating system is particularly interesting in that it does not let any vacuum spaces build between the sectors and the massage obtained is complete, uniform and effective. Legs and arms strap of **Linfopress Easy** are made of a special plastic material, extremely resistant and very soft, in order to avoid rubescence.

The appliance can pilot a single applicator or two applicators by the specific splitters; the leg straps allow to use a special two-sectors abdominal strap by means of an extra connection to the inguinal sector.

Linfopress Easy works in a very easy way. Just put on the leg or arm strap and start the appliance by pressing the power supply switch. The treatment cycle consists in inflating the four sectors one after the other with 25 seconds' time and 5 seconds' deflating. The cycle is repeated until the user switches the appliance off from the main switch. The pressure in the various sectors is adjusted by the user by acting on the knob on the front panel.

Linfopress Easy permits to act on venostasis and limphostasis, with a homogeneous and constant massage; it is useful in treating lymphoedemas and various kind of oedema; furthermore it is suitable for short run after a long physical effort in sport and to fight against the blemishes of cellulitis.